

ISI Alumi-Hand™ and DuraHand™ – Surgical Hand Immobilizers

Disposable or Reusable...it's up to you!

The Alumi-Hand is the unique ready-to-use sterile, disposable hand immobilizer. The DuraHand is our new reusable hand immobilizer. Both hand immobilizers are lead-free, Latex-free, and X-ray compatible. And of course, both are priced to be a value.

Alumi-Hand and DuraHand are easy to use.

1. Place the patient's hand on top of the Alumi-Hand or DuraHand.
2. Bend the wrist tabs over the patient's wrist and bend the immobilizer's finger tips over the patient's fingers, as needed. The patient's hand should then be immobilized.
3. After the procedure, straighten the immobilizer's fingers and wrist tabs.
4. Alumi-Hand: If the Alumi-Hand was used, dispose of it according to hospital waste and recycling instructions.

DuraHand: If using the DuraHand, straighten the DuraHand's finger and wrist tabs and reprocess according to the sterilization instructions included with the DuraHand.

Ordering Information

Large Alumi-Hand CE#0120

Stock No.: 760

Dimensions: 9 1/2" x 8 1/2"

Packaged in individual sterile pouches, 4 per box

Extra-Large Alumi-Hand CE#0120

Stock No.: 761

Dimensions: 13" x 10 1/2"

Packaged in individual sterile pouches, 3 per box

Large DuraHand CE

Stock No.: DH-225

Dimensions: 9 1/2" x 8 1/2"

Packaged 1 per box; NON-STERILE

Extra-Large DuraHand CE

Stock No.: DH-350

Dimensions: 13" x 10 1/2"

Packaged 1 per box; NON-STERILE



The Alumi-Hand is the only sterile, disposable hand immobilizer.



The DuraHand is our new reusable hand immobilizer.



Large shape
9 1/2" x 8 1/2"



Extra-large shape
13" x 10 1/2"



**ISI Medical &
O.R. Products**

A DIVISION OF INSTRUMENT SPECIALISTS, INC.

www.isisurgery.com

Toll Free in the USA: (800) 537-1945

Tel: (830) 249-9535 • Fax: (830) 249-9433

32390 IH-10 West, Boerne, Texas 78006-9214