



Instrument Specialists, Inc.
32390 IH 10 West
Boerne, Texas 78006
USA

Inside the USA: 800-537-1945
Outside the USA: +1-830-249-9535
Fax: +1- 830-249-9433
www.ISIsurgery.com

**ISI Universal
K Table**
Upper Extremity,
Hand Surgical Table
REF: K3510

**ISI Advanced Universal
K Table**
Upper Extremity,
Hand Surgical Table
REF: AUT-5000



Rx Only



How to Use: Attach the table to the rail of the OR table or stretcher and tighten the clamps to prevent the table from moving or becoming detached from the table during a procedure. Adjust table leg to the desired height and secure by tightening the knob on the inside of the leg. The table must be draped prior to any procedures.




This table is not safe to use without the support of the adjustable leg.

You may also use ISI's Traction System, Stretcher Mount or Under Pad Mounting Bracket for additional uses. Contact ISI for pricing and stock number information on all items below.




K-3650
K-700
PD-200
PD-250
K-300ST

Traction System
Under Pad Mounting Bracket
2" Pad Contoured Foam Pad
4" Pad Contoured Foam Pad
Stretcher U Bar

Cleaning Recommendations: The Universal K Table, and the products listed above, may be cleaned with any standard disinfectant. After cleaning, store in accordance with standard hospital operating procedures to maintain cleanliness. Inspect before and after each use.

 If the table is not properly assembled or is damaged, **DO NOT USE!** Contact ISI for assistance with any necessary repairs or replacement parts. Maximum weight capacity is 50lbs (22kg).

Explanation of Symbols

Symbol	Explanation
	Manufacturer
SN	Serial Number
REF	Catalogue Number
	Caution, Consult Accompanying Documents
	Non-Sterile