



Instrument Specialists, Inc.  
 32390 IH 10 West  
 Boerne, Texas 78006  
 USA

Inside the USA: 800-537-1945  
 Outside the USA: +1-830-249-9535  
 Fax: +1- 830-249-9433  
[www.ISIsurgery.com](http://www.ISIsurgery.com)

## ISI Self Supporting Arm Table

Upper Extremity, Legless  
Surgical Table  
**REF: AT-4000**

## ISI Ultra-Light Table


Upper Extremity, Legless  
Surgical Table  
**REF: UL-8000**



**Rx Only**




**How to Use:** Attach the table to the rail of the OR table or stretcher and tighten the clamps to prevent the table from moving or becoming detached from the table during a procedure. The table must be draped prior to any procedures.

 When using this table without the Standard Leg make certain that the rail clamps are locked down tight to prevent the table from moving during procedures.




You may also use ISI's Traction System, Stretcher Mount, Under Pad Mounting Bracket, or Standard Single Post Leg for additional uses. Contact ISI for pricing and stock number information on all items below.

K-3650	Traction System
PD-100	2" Pad Contoured Foam Pad
PD-150	4" Pad Contoured Foam Pad
SL-2001	Standard Single Post Leg
K-708	Under Pad Mounting Bracket (for AT-4000)
K-700	Under Pad Mounting Bracket (for UL-8000)

**Cleaning Recommendations:** The Self Supporting Arm Table or Ultra-Light Table, and the products listed above, may be cleaned with any standard disinfectant. After cleaning, store in accordance with standard hospital operating procedures to maintain cleanliness. Inspect before and after each use.

 If the table is not properly assembled or is damaged, **DO NOT USE!** Contact ISI for assistance with any necessary repairs or replacement parts. Maximum weight capacity is 50lbs (22kg).

### Explanation of Symbols

Symbol	Explanation
	Manufacturer
<b>SN</b>	Serial Number
<b>REF</b>	Catalogue Number
	Caution, Consult Accompanying Documents
	Non-Sterile