



Instrument Specialists, Inc.  
32390 IH 10 West  
Boerne, Texas 78006  
USA

Inside the USA: 800-537-1945  
Outside the USA: +1-830-249-9535  
Fax: +1- 830-249-9433  
[www.ISIsurgery.com](http://www.ISIsurgery.com)

## Carter Hand Surgery Table


REF: CST-2007



Rx Only



**How to Use:** Attach the table to the rail of the OR table by pulling the release lever on the mounting bracket. Once the bracket is seated onto the rail, release the lever and it will capture the rail in the plastic components. Adjust table leg to the desired height and secure by tightening the knob on the inside of the leg. The table must be draped prior to any procedures.

 If the table is not properly assembled or is damaged, **DO NOT USE!** Contact ISI for assistance with any necessary repairs or replacement parts. Maximum weight capacity is 50lbs (22kg).

**Cleaning Recommendations:** The Carter Table and the products may be cleaned with any standard disinfectant. After cleaning, store in accordance with standard hospital operating procedures to maintain cleanliness. Inspect before and after each use.

**REMOVE THE NYLON FINGERTRAPS AND STERILIZE THE TRACTION BAR SEPERATELY FROM THE FINGER TRAPS FOLLOWING THE INSTRUCTIONS BELOW**

### Sterilization Instructions: TRACTION BAR

1. Sterilize for 4 minutes at 132°C gravity unwrapped (Flash Sterilization).  
OR
2. Sterilize for 4 minutes at 132°C pre vacuum wrapped.

The above cycles are considered standard for hospital steam sterilization. By following either of these guidelines for sterilization, tests have shown that the user can expect a 10<sup>-6</sup> SAL (Sterilization Assurance Level).

## Sterilization instructions: NYLON FINGER TRAPS

### 1. Steam Sterilization General Instructions:

Follow standard hospital sterilization practices

- A. Place clean instruments in an instrument tray
- B. If instruments are to be wrapped, two double thicknesses of #140 thread count wrappers should be utilized. Exposure times are the same for wrapped or unwrapped instruments
- C. Follow the instructions for the gravity air displacement steam sterilization listed below

### 2. Gravity Steam Sterilization:

- A. Double wrap in standard central supply wrap.
- B. Set the temperature and corresponding exposure time:  
121-123 degrees Celsius for 30 minutes

After cleaning, store in accordance with standard hospital operating procedures to maintain cleanliness. Inspect finger trap before and after each use.



If the Finger Trap is split or frayed, **DO NOT USE**. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

## Explanation of Symbols

Symbol	Explanation
	Manufacturer
<b>SN</b>	Serial Number
<b>REF</b>	Catalogue Number
	Caution, Consult Accompanying Documents
	Non-Sterile