

C-Thru Adult Armboard

Upper Extremity Surgical Armboard/Hand Table



(Shown with optional leg and 2” pad)

How to Use: Slide the longer end of the ArmBoard completely under the pad of a stretcher, gurney or OR Surgical Table. (The pad of the stretcher may need to be lifted or removed and then put back in place once assembly is complete.) The Arm-Board will be held in place by the weight of the patient.

The optional 1” or 2” pad may be placed on the surgical field area and draped prior to procedures if a higher platform is preferred.

To disassemble, the patient should be removed from the stretcher or table and pull the ArmBoard and pad from the surgical field side.

You may also use the optional single post C-Thru Leg for more stability. Contact ISI for stock number and information.

Cleaning Recommendations: The C-Thru Adult ArmBoard and pads may be cleaned with any standard disinfectant without ammonia. After cleaning, store in accordance with standard hospital operating procedures to maintain cleanliness. Inspect before and after each use.