



Instrument Specialists, Inc.  
32390 IH 10 West  
Boerne, Texas 78006  
USA

Inside the USA: 800-537-1945  
Outside the USA: +1-830-249-9535  
Fax: +1-830-249-9433  
[www.ISIsurgery.com](http://www.ISIsurgery.com)

## Traction System

REF: K-3650, K-3750



Rx Only



The ISI Traction System may be used with ISI Hand Tables.

**How to Use for a Sterile Procedure:** After the patient's hand is prepped and stable, the technician can then attach the Finger Traps and traction chains. Attach an S-hook or hooks and attach the necessary weights.



Check that the patient is secured before proceeding with operating procedure.

**Cleaning Instructions:** ISI's Traction Bar is Stainless Steel and should be cleaned following standard hospital cleaning procedures.

After cleaning, store in accordance with standard hospital operating procedures to maintain cleanliness.

### **1. Steam Sterilization General Instructions:**

Follow standard hospital sterilization practices

- A. Place clean instruments in an instrument tray
- B. If instruments are to be wrapped, two double thicknesses of #140 thread count wrappers should be utilized. Exposure times are the same for wrapped or unwrapped instruments
- C. Follow the instructions for the gravity air displacement steam sterilization listed below

**2. Gravity Steam Sterilization:**

- A. Double wrap in standard central supply wrap.
  - B. Set the temperature and corresponding exposure time:  
121-123 degrees Celsius for 30 minutes
- 
- 1. Sterilize for 4 minutes at 132-135°C gravity unwrapped (Flash Sterilization).
- OR
- 2. Sterilize for 4 minutes at 132-135°C pre vacuum wrapped.

**Dry time: 15 – 30 minutes**




The above cycles are considered standard for hospital steam sterilization. By following either of these guidelines for sterilization, tests have shown that the user can expect a 10-6 SAL (Sterilization Assurance Level).

You may also use ISI's Digi Grip Traction System and weight set for additional patient stability. Contact ISI for pricing and stock number information on all sizes of Stainless Steel Finger Traps.

DG-3000	Digi Grip
S Hook	Stainless Steel S Hook
0350	½ pound (.023 Kilogram) weight
0352	1 ½ pound (.068 Kilogram) weight
0354	2 ½ pound (1.13 Kilogram) weight
0357	5 pound (2.27 Kilogram) weight

 If the Traction Bar is damaged, **DO NOT USE**. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

**Explanation of Symbols**

Symbol	Explanation
	Manufacturer
<b>REF</b>	Catalogue Number
	Caution, Consult Accompanying Documents
	Non-Sterile